



LOKI

INJECTION GUIDELINES

FOR DOG – VETERINARY USE ONLY

Implant's position



① Unclip the holder:



② Put **BSS** inside all the cartridge, on the IOL and on the blue silicone.
Wait at least 1 minute



③ Put **viscoelastic product** inside the tip of the cartridge, in front of and on the IOL




④ With **forceps**, **press down on the IOL** and close the wings in the same time "click":



⑤ Waiting position before injection:


Push the blue piston, it **must arrive at Position 1** without cross the line.

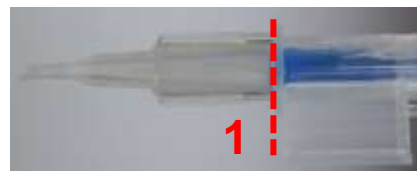

Keep the product at this state until final injection

⑥ During injection :

Push the blue piston until **Position 2**

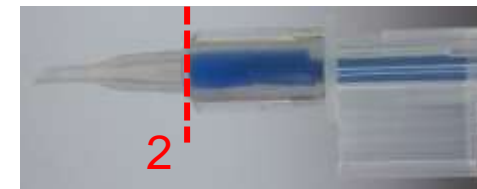

Immediately, put the cartridge inside incision of the eye.

Inject in such a continuous way until the lens exit